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Instructions for Use – Please Read All Instructions Before Use.

Attaching the Leg Extensions (if not attached already)

- Place frame on its side on a level surface.
- Attach leg extensions with suction feet to the frame on the same side as where the support arm will be. Attach leg extensions with flange tips to frame on opposite side of frame.
- The transfer bench legs with suction feet should be the two legs that are to be placed inside the tub.

Attaching the Backrest

- Determine which direction the user will face when sitting on the bench in the tub.
- Position the bench on a level surface.
- Place the front seat Z-clips under the front seat support tube.
- Lower the seat back and snap the back seat clip into place.

CAUTION: Backrest is not designed to support the full weight of the user.

Attaching the Support Arm

- To attach the support arm, position the seat arm against the leg frame and slide the armrest tubing into the tube openings. Make sure that the push buttons fully protrude through the holes in the bracket.
NOTE: An audible “click” will be heard.
- To remove the seat arm, depress the push buttons and pull up.

NOTE: The end with the Support Arm and Suction Feet is the side that should be inside the tub.

Adjusting Leg Height

- Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
NOTE: An audible “click” will be heard.
- Repeat for other 3 legs.

CAUTION: Ensure that all four legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

Using Your Transfer Bench

NOTE: Be sure to move slowly throughout use to prevent injuries.

NOTE: Children or those who need assistance should NOT be left unattended during use of this product.

From a seated position (such as a wheelchair):

- Place your wheelchair next to the bench, facing the front of the bathtub.
- Engage wheelchair locks.
- Transfer your torso to the transfer bench first, and then lift your legs into the tub. Move slowly to prevent injuries.
- After bathing, lift your legs out of the tub first, then your torso.

From a standing position:

- Sit on the side of the transfer bench that is outside of the tub.
- Extend your arms to grab the support arm of the bench and slide your body over onto the bench. Lift each leg over the tub slowly.
- Position yourself to a comfortable position in center of bench.
- After bathing, lift your legs out of the tub first, then your torso slowly.

Features:

- Provides assistance to those who have difficulty stepping over bathtub walls
- Comfortable, cushioned seat and backrest are water-tight and easy to clean
- Anodized aluminum frame is lightweight, durable and rust-resistant
- Side support arm aids in lateral transfer
- Seat height is adjustable in 1" increments
- Reversible backrest to accommodate any bathroom
- Attractive platinum grey color
- Comes assembled with tool-free arm and back attachments
- Available with or without commode pail and cover - 7956A and 7958A comes with commode pail and cover
- Available with additional arm rest – Item 7958A only
- 300 lb. maximum weight capacity
- Designed to meet the requirements of HCPCS code: E0247
- Lifetime limited warranty

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© 2009, GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, Georgia 30360. Made in China. www.grahamfield.com

Specifications subject to change without notice
Especificaciones son sujetas a cambio sin aviso

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Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex products listed on the cover of this manual for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care

Parts covered under the lifetime warranty include: (frame, welding, screws, frame locking components). Specific items "excluded" from this warranty of any kind are: "ALL" other parts not specifically listed above. The warranty does not extend to non-durable components, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

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