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Instructions for Use – Please Read All Instructions Before Use. Setup

1. Remove frame, seat, lid, pail, cover, and splash guard from carton.
2. Attach leg extensions (if not attached already).
3. Attach seat and then lid to rear support of frame. Place slash shield or pail and cover in frame.
NOTE: When installing the seat onto the frame, make sure that the seat is evenly spaced between the side frames to ensure proper stability.
NOTE: Push down firmly until the seat snaps into place.
NOTE: DO NOT use excessive force when installing the seat and lid.

Adjusting Leg Height (2190A and 2195A-4)

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
NOTE: An audible “click” will be heard.
2. Repeat for other 3 legs.
CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

Inserting Back Bar

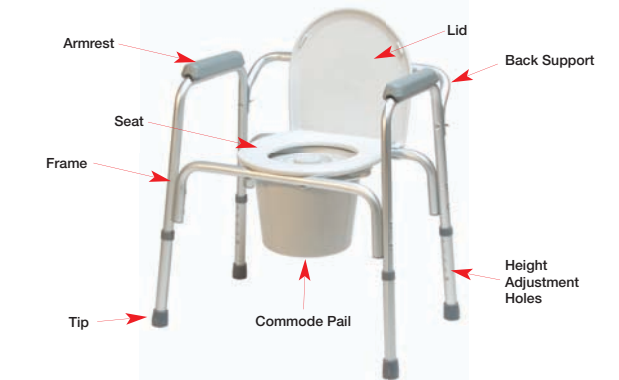
1. Remove back bar from carton.
2. Insert back bar into channel guides on back of commode until back bar push buttons protrude fully through channel guide holes (you will hear an audible click).

Using Commode over Toilet

1. Set commode to desired height (described in previous steps).
2. Remove commode pail and lid.
3. Position splash guard in place onto pail supports.
4. Remove back support (described in previous steps).
5. Lift toilet bowl seat.
6. Place commode over toilet with slash shield positioned over the center of the toilet bowl. Commode is now ready for use. Clean seat and splash guard with disinfectant after each use.

Using as Bedside Commode

1. Set commode to desired height (described in previous steps).
2. Lift commode seat and lid and place Commode pail onto pail supports. Ensure that pail is properly positioned below commode opening.
3. Place commode seat down. Commode is now ready for use.
4. When use is finished using commode, place commode pail lid over commode pail to contain odors.
5. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.

- Features – 2190A, 2195A-4, and 2215A-2**
- 3-in-1 design is versatile and economical
 - Aluminum frame is light weight and easy to transport
 - Sturdy plastic seat snaps-on easily
 - Comes complete with commode pail, cover and splash guard
 - Contoured armrests enhance user comfort
 - 4" rear swivel, locking casters (2215A-2 only)
 - Limited Lifetime Warranty

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Capacidad de Peso
2215A-2 y 2195A-4 – 300lbs
2190A – 400lbs

© 2009, GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, Georgia 30360.
Made in China. www.grahamfield.com

Specifications subject to change without notice
Especificaciones son sujetas a cambio sin aviso

GF0700013RevB09

Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: Users with limited physical capabilities should be supervised or assisted when using the commode.

WARNING: This product has a maximum weight capacity of 300lbs

WARNING: GF Health Products Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

WARNING: Leg extensions with rubber tips MUST be in contact with the floor at all times.

WARNING: Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

WARNING: Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.

WARNING: Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

WARNING: The commode seat MUST be installed before sitting on the commode.

WARNING: If components are damaged or missing, contact your GF Health Products distributor immediately.

Three-In-One Aluminum Commode

Inodoro de Aluminio Tres-en-Uno
Items 2190A, 2195A-4 & 2215A-2

User Instructions and Warranty Information

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Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex products listed on the cover of this manual for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care

Parts covered under the lifetime warranty include: (frame, welding, screws, frame locking components). Specific items "excluded" from this warranty of any kind are: "ALL" other parts not specifically listed above. The warranty does not extend to non-durable components, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

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