

Note: The most current version of these instructions can be found online at www.grahamfield.com


- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the Raised Toilet Seat with Locking Clamp. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the raised toilet seat is not properly assembled and installed, personal injury and damage to the raised toilet seat could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the raised toilet seat.**
- ⚠ **WARNING: This product has a weight limit of 300 lb.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**



6486A



6487RA

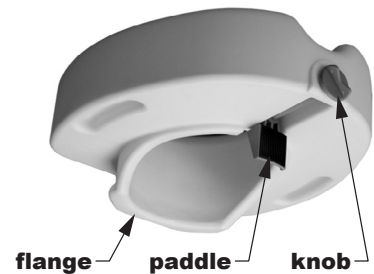

INSTALLATION

1. Remove toilet seat from carton and protective packaging.
2. If installing Model 6486A, proceed to step 3.
If installing Model 6487RA, follow steps a-e to install arms, then proceed to step 3.
 - a. Remove arms from box.
 - b. Insert each end of arm into the raised toilet seat as shown in center picture above.
 - c. Push arm downward until you hear it click into place.
 - d. Pull arm upward slightly to secure proper installation. If loose, repeat step c.
 - e. Repeat steps b-d for other arm.
3. Lift lid and seat of toilet. This raised toilet seat fits most current, standard toilets.
4. Place raised toilet seat on toilet bowl rim, hooking rear flanges under rear edge of bowl. Ensure paddle is back far enough to clear inner rim of bowl. If not, turn knob counterclockwise.
5. Ensure raised toilet seat rests flat on toilet bowl rim; straighten unit on bowl rim.
6. To tighten, rotate adjustment knob clockwise. Push unit back against rear of toilet rim while tightening knob. When you feel resistance, turn an additional $\frac{1}{8}$ turn to snug. If top rear of unit hits the toilet seat or lid, you must remove the seat and lid for proper installation.

Note: The adjustment knob head must be flush with the front of the toilet seat for proper fit. On smaller bowls, if paddle is moved to rear of its travel, the knob may pop out of its original position; the knob will automatically re-engage when you re-tighten.

Note: Do not force the knob to turn. If necessary, jiggle the knob a little to start the threads.

7. The raised toilet seat should now be securely installed on the toilet bowl. To ensure that you have a proper fit, intentionally shake the raised toilet seat to ensure proper placement and fit. The raised toilet seat should not move; if it does, re-install.

⚠ WARNING: Ensure that the raised toilet seat is installed as described. If the raised toilet seat is not properly assembled and installed, personal injury and damage to the raised toilet seat could result.

REMOVAL

1. Rotate knob counter-clockwise until raised toilet seat is free.
2. Lift front of raised toilet seat, then remove from toilet bowl.

OPERATION

⚠ WARNING: Always ensure the raised toilet seat is securely installed before use.

MAINTENANCE

⚠ WARNING: Check the raised toilet seat weekly for signs of damage or cracking. If damage or cracking is found, DO NOT use the raised toilet seat. Contact your Lumex dealer immediately.

⚠ WARNING: Inspect the raised toilet seat weekly and tighten it as needed to ensure secure installation.

▲ Caution: Never use abrasive cleaners or scouring pads on any part of the raised toilet seat.

- To clean the raised toilet seat, wash it with soap and water or a mild detergent, then rinse it with water. Ensure that the raised toilet seat is rinsed well before use.
- To clean interior of raised toilet seat, flush with warm soapy water, then drain and allow to air dry for several hours.
- If moisture accumulates inside raised toilet seat, do the following:
 - a. Remove raised toilet seat from toilet bowl (see above instructions).
 - b. Turn raised toilet seat upside down and tilt toward drain opening.
 - c. Rotate until drain opening is facing downward and moisture drains.
 - d. Allow raised toilet seat to air dry for several hours before re-installation.

WARRANTY

GF Health Products, Inc. warrants the Lumex 6486A/6487RA Raised Toilet Seat with Locking Clamp as follows:

Limited lifetime warranty against manufacturer's defects. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.



GRAHAM-FIELD.

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice. Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© February 2009 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0601 • www.grahamfield.com