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Instructions for Use – Please Read All Instructions Before Use.

Setup

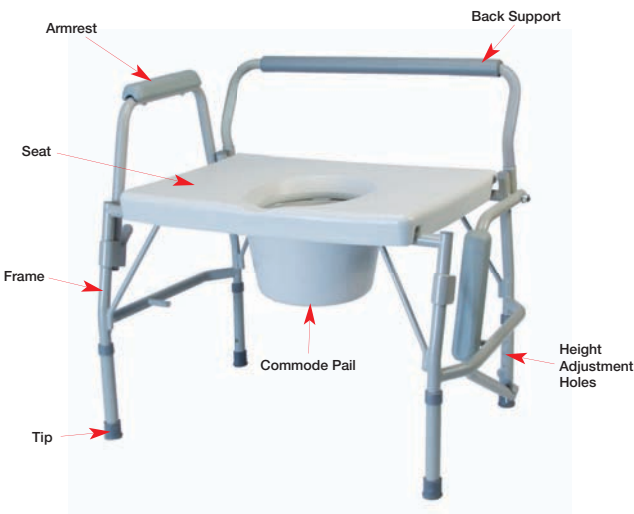
1. Remove frame, commode platform and back support from box. Unwrap all packaging materials.
2. Commode platform is already attached to frame, as shown below; ensure that all sides of platform are secured to frame
3. The commode comes in its lowest position. Adjusting to desired height is recommended before use.

Arm Adjustment

- **Drop Armrest:** to drop the armrest, push on the release lever inwards to release the armrest. This releases the armrest allowing for the transfer on and off the commode.
- **Raise Armrest:** to raise the armrest, lift the armrest to the upright position, ensuring that the push button protrudes fully from locking hole.

Adjusting Leg Height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
NOTE: An audible “click” will be heard.
2. Repeat for other 3 legs.
CAUTION: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

Features

- Arms drop independently below seat level for safe lateral transfers (6438A only)
- Arms automatically lock in raised position
- Extra wide platform seat is comfortable for long periods of sitting
- Welded steel construction and extra wide base provide added stability
- Cushioned armrests enhance user comfort
- Comes complete with commode pail, cover and splash guard
- Removable back bar facilitates use as a bedside commode, raised toilet seat or toilet safety frame.
- 6438A – 600lb maximum weight capacity
7446A-2 – 400lb maximum weight capacity
- Designed to meet the requirements of HCPCS code: E0168

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© 2009, GF Health Products, Inc., 2935 Northeast Parkway, Atlanta, Georgia 30360.
Made in China. www.grahamfield.com

Specifications subject to change without notice
Especificaciones son sujetas a cambio sin aviso

GF0900148RevA09

Maintenance

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners, or scouring pads on any part of the product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your GF Health Products, Inc authorized distributor.

For more information regarding GF Health Products, Inc. products, parts and services, please contact your distributor or visit us online at www.grahamfield.com.

Safety Guidelines – Please Read Before Use

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

WARNING

WARNING: DO NOT use this product without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING: Users with limited physical capabilities should be supervised or assisted when using the commode.

WARNING: Item 6438A has a maximum weight capacity of 600lbs
Item 7446A-2 has a maximum weight capacity of 400lbs

WARNING: GF Health Products Inc. assumes no responsibility for any damage or injury caused by improper installation, assembly, or use of this product.

WARNING: Leg extensions with rubber tips MUST be in contact with the floor at all times.

WARNING: Ensure that the push buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

WARNING: Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.

WARNING: Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them IMMEDIATELY if any of these conditions exist.

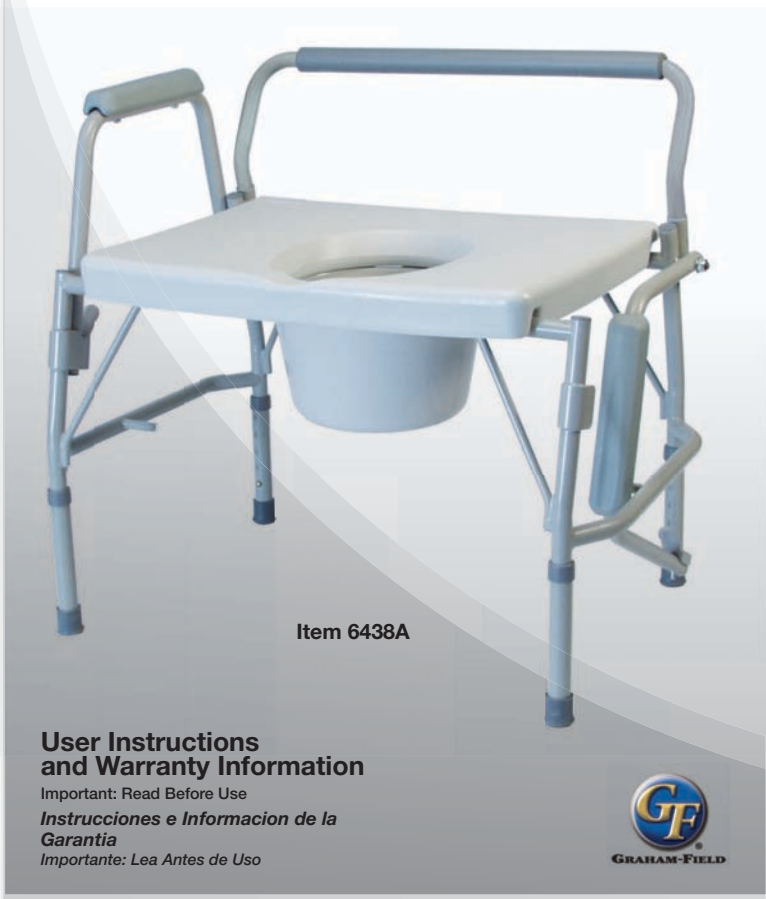
WARNING: The commode seat MUST be installed before sitting on the commode.

WARNING: If components are damaged or missing, contact your GF Health Products Inc. distributor immediately.

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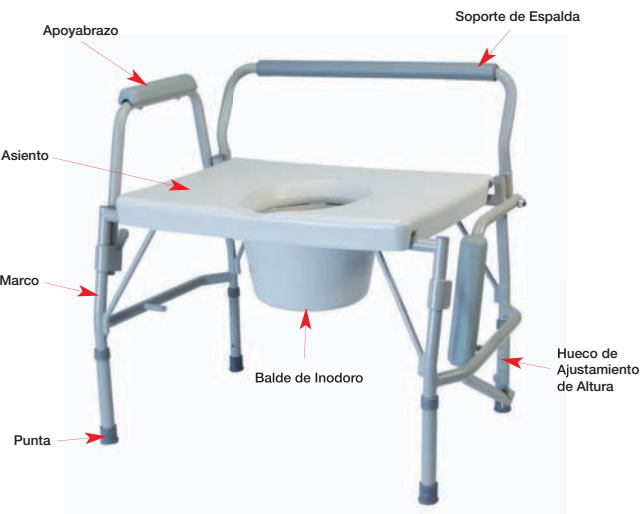
Lifetime Limited Warranty

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products. We are dedicated to providing products that will give our customers years of dependable service.

GF Health Products, Inc. hereby warrants specific parts of the Lumex products listed on the cover of this manual for the lifetime of the product against defective workmanship and/or material defects when used for the purpose intended, under normal conditions, and provided the product receives proper care

Parts covered under the lifetime warranty include: (frame, welding, screws, frame locking components). Specific items "excluded" from this warranty of any kind are: "ALL" other parts not specifically listed above. The warranty does not extend to non-durable components, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized GF Health Products, Inc. Distributor.

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