Model Nos: 7701C,7703C,7711C,7713C,7720C,7721C,
7723C,7724C,7725C,7726C,7727C,7728C,5652C
7704C,7706C,7655C,7657C,7544C

Maximum User Weight: 127kg (20st)
(Do not exceed this weight)

Model Nos: 7653C7665C7674C
9890C9889C9888RC 9888LC

Maximum User Weight: 100kg (16st)
(Do not exceed this weight)

USING STICKS

Please read all instructions before use.

PLEASE NOTE:

These are our recommendations but should be checked with your physiotherapist.

CORRECT ADJUSTMENTS

When the stick is measured correctly the user should be able to maintain an upright posture with the elbow slightly flexed. In this way, body weight is taken through the stick by pushing down on the stick when walking. If adjustable, your stick will have either a horseshoe or spring-pin type adjustment as shown below.

Horseshoe Clip Adjustment

1. Remove the clip by grasping it between thumb and forefinger and pulling out.
2. Slide the leg in or out to the ideal position.
3. Push the clip back firmly into the hole.
4. Check the clip is fully located before use.

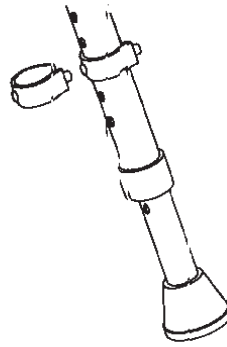
FIG. 1

Spring Pin Adjustment

1. Press the two buttons in.
2. Push the leg as if to shorten.
3. Turn the leg slightly.
4. Extend the leg to the required position, turn and re-locate buttons into the appropriate holes.
5. Check pins are fully located before use.

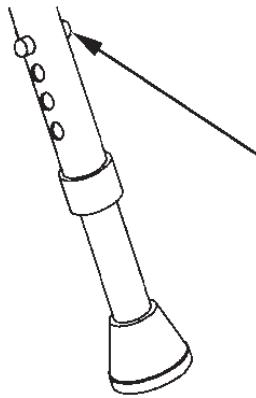
FIG. 2

WALKING WITH THE STICK


Walk in a normal manner placing the stick on the ground at the same time as the affected leg.

Remember the stick should normally be held in the hand opposite the affected leg

CHAIRS


Getting Up:

The stick should be near the arm of the chair. Lean forward and with the hands on the arm of the chair, push forward and stand up using the stick for support.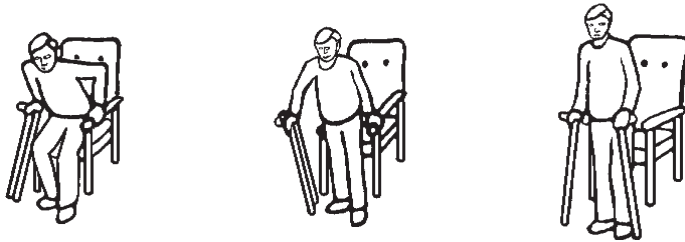

Sitting Down:

When returning to the chair, turn round, feel the chair behind the legs, place your hands on the chair arms, bend forward and lower yourself gently into the chair. The stick can be held or rested nearby.

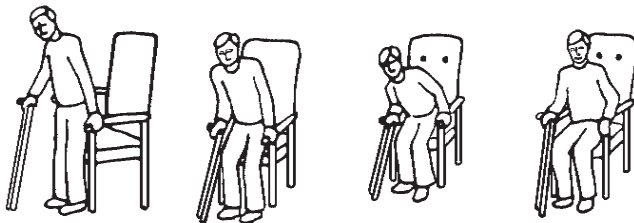CONTINUED ➡

GENERAL SAFETY

- 1 Use a chair with high seat and arms to help you sit and stand up with ease.
- 2 Remove obstacles such as loose rugs, draught excluders and worn floor coverings.
- 3 Ensure lighting in hallways and stairways is adequate.
- 4 Wear flat supportive shoes.
- 5 Avoid wet floors.
- 6 Beware of outdoor hazards including uneven surfaces, wet leaves and ice.
- 7 Do not store this product in temperatures below freezing.
- 8 Take special care if carrying a bag or similar package.
- 9 All equipment should be regularly inspected for any damage or significant signs of wear.

Check for:

Bent or damaged tubes or cracks at the tube joints.

Both buttons of the spring clips are present or horseshoe clips are not loose or worn.

Elongated adjustment holes.

Loose handles.

Smooth, worn or split ferrules (rubber tips).

Replace where necessary.

- 10 Regular cleaning with a mild disinfectant is recommended.
DO NOT use bleach or any solvents (such as lighter fluid).

Sunrise Medical Limited

High Street, Wollaston, West Midlands, DY8 4PS England
Tel + 44 (0) 01384 446688 Fax + 44 (0) 01384 446699

Conforms to
The Medical Device Directive 93/42/EEC 1993

Modelli n.: 7701C,7703C,7711C,7713C,7720C,7721C,
7723C,7724C,7725C,7726C,7727C,7728C,5652C,
7704C,7706C,7655C,7657C,7544C

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