Fix Front

Swing Away

Neon

Directions for use
Notice D'Utilisation
Manual De Usuario
Manuale Per l'Uso
Gebruikershandleiding
Manual do utilizador

QUICKIE®

INNOVATE.
BUILD FOR THE FUTURE

Foreword (1.1.1)

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL. This user's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

Maintaining **close links** with our customers is of great importance to us at Sunrise Medical: we would therefore like to keep you up-to-date with the latest developments. Keeping close to our customers also means fast service when you need replacement parts or accessories, or just have a question about your wheelchair – with as little fuss as possible.

We want you to be satisfied with our products and service. Sunrise Medical therefore constantly strives to continuously develop its products. For this reason, changes can occur in our range of products with regard to shape, technology and fittings. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

SUNRISE MEDICAL has been awarded the ISO- 9001 Certificate, which affirms the quality of our products at every stage, from research and development to production.

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance or safety of your wheelchair.

As the manufacturer, SUNRISE MEDICAL, declares that the lightweight wheelchairs conform to the 93/42/EEC / 2007/47/EEC guideline.

If there is no authorised dealer in your area or you have any questions, you can contact Sunrise Medical either in writing or by telephone at:

Sunrise Medical Ltd.
Sunrise Business Park
High Street, Wollaston
West Midlands DY8 4PS
England
 Telephone +44/1384-446622
 Fax +44/1384-446644
 www.sunrisemedical.com

IMPORTANT:
 DO NOT USE YOUR WHEELCHAIR UNTIL THIS
 MANUAL HAS BEEN READ AND UNDERSTOOD.

Table of contents

Foreword for Wheelchairs	
General safety notes and driving restrictions	4-5
Overview	5
Transportability	6+7
Handling for transportation	
Quick-release axles on rear wheel	8
Options	
Tipping tubes	9
Brakes	8+9
Suspension system	9
Footboard	9
Footboard adjustment	10
Footrests	10
Headrest	10
Castor	10
Seat	11
Wheelbase extension	11
Wheel tracking	11
Backrest	11
Sideguard	12
Push handles	13
Lap belt instructions	13
Anti-tip tubes	14
Seat sling	14
Crutch holder	14
Therapy table	14
Stabilising bar	14
Tyres and mounting	15
Possible problems	15
Maintenance	15
Nameplates/Guarantee	15
Technical Data	16
Torque	16

Use

Wheelchairs are exclusively for a user who is unable to walk or has limited mobility, for their own personal use in- and outdoor.

The maximum weight limit (includes the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the crossbar or stabiliser bar below the seat.

Warranty can only be taken on if the product is used under the specified conditions and for the intended purposes.

The expected life of the wheelchair is 5 years.

Please **DO NOT** use or fit any 3rd party components to the wheelchair unless they are officially approved by Sunrise Medical.

Area of application

The varieties of fitting variants, as well as the modular design, mean that it can be used by those who cannot walk or have limited mobility because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect deformity
- Joint contractures/joint injuries
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia as well as for elderly people who still have strength in the upper body.

When considering provision, please also note the body size, weight, physical and psychological constitution, the age of the person, living conditions and environment.

General safety notes and driving restrictions (3.2)

The engineering and construction of this wheelchair has been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded in parts. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must be strictly observed.

Unprofessional or erroneous changes or adjustments increase the risk of accident. As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else. We would like to remind you that you are therefore also subject to any and all traffic laws.

Be careful during your first ride in this wheelchair. Get to know your wheelchair.

Before each use, the following should be checked:

- Quick-release axles on the rear wheels
- Velcro on seats and backrests
- Tyres, tyre pressure and wheel locks.

Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the user's manual.

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill.

When riding over a step or up an incline frontally, the body should be leaning forward.

Danger!

NEVER exceed the maximum load of 125 kg for driver plus any items carried on the wheelchair. If you exceed the maximum load, this can lead to damage to the chair, or you may fall or tip over, lose control and may lead to serious injury of the user and other people.

Danger!

When it is dark, please wear light clothing or clothing with reflectors, so that you can be seen more easily. Make sure that the reflectors on the side and back of the wheelchair are clearly visible. We would also recommend that you fit an active light.

Danger!

To avoid falls and dangerous situations, you should first practice using your new wheelchair on level ground with good visibility.

Danger!

When getting on or off the wheelchair, do not use the footboards. These should be flipped up beforehand and swung to the outside as far as possible. Always position yourself as close as possible to the place where you wish to sit.

Danger!

Only use your wheelchair properly. For example, avoid travelling against an obstacle without braking (step, kerb edge) or dropping down gaps.

Danger!

The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheel locks to prevent such rolling. Always apply both wheel locks; otherwise, your wheelchair could tip over.

Danger!

Explore the effects of changing the centre of gravity on the behaviour of the wheelchair, for example on inclines, slopes, all gradients or when overcoming obstacles. Do this with the secure aid of a helper.

Danger!

With extreme settings (e.g. rear wheels in the most forward position) and less than perfect posture, the wheelchair may tip over even on a level surface.

Danger!

Lean your upper body further forward when going up slopes and steps.

Danger!


Lean your upper body further back when going down slopes and steps. Never try to climb and descend a slope diagonally.

Danger!

Avoid using an escalator which may lead to serious injury in the event of a fall.

Danger!

Do not use the wheelchair on slopes $> 10^\circ$. The Dynamic safe slope is dependant on the chair configuration, the users abilities and the style of riding. As the users abilities and style of riding cannot be pre-determined then the max safe slope cannot be determined. Therefore this must be determined by the user with the assistance of an attendant to prevent tipping. It is strongly recommended that inexperienced users have Ant-tips fitted.

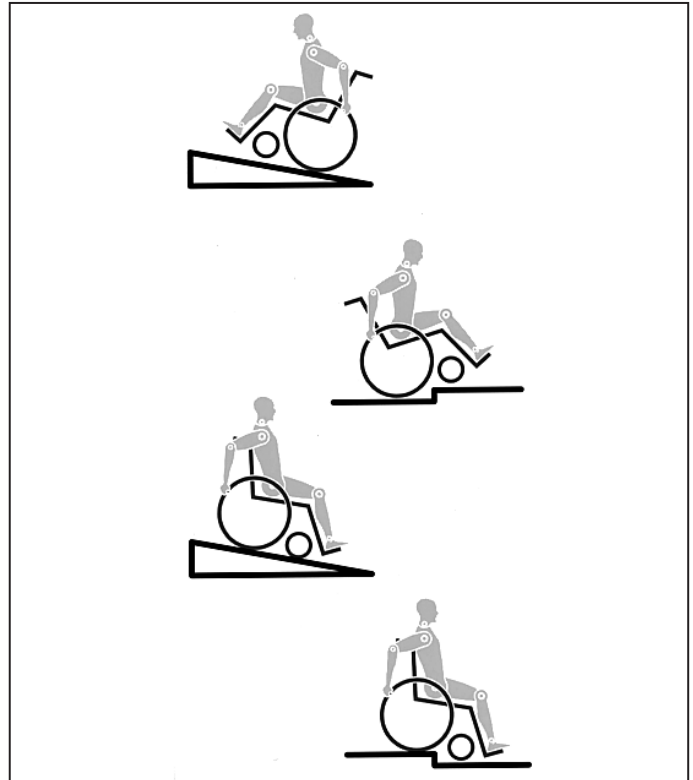Danger!

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill.

Danger!

Do not use your wheelchair on muddy or icy ground. Do not use your wheelchair where pedestrians are not allowed.

Danger!

To avoid hand injuries do not grab in between the spokes or between the rear wheel and wheel lock when driving the wheelchair.

Danger!

In particular when using lightweight metal handrims, fingers will easily become hot when braking from a high speed or on long inclines.

Danger!

Only attempt stairs with the help of an attendant. There is equipment available to help you, e.g. climbing ramps or lifts, please use them. If there is no such equipment available, then the wheelchair must be tipped and pushed, never carried, over the steps (2 helpers). We recommend that users over 100 kg in weight do not use this stairway manoeuvre!

In general, any anti-tip tubes fitted must be set beforehand, so that they cannot touch the steps, as otherwise this could lead to a serious tumble. Afterwards the anti-tip tubes must be set back to their correct position.

Danger!

Make sure that the attendant only holds the wheelchair using securely mounted parts (e.g. not on the footrests or the sideguards).

Danger!

When using the lifting ramp make sure that the anti-tip tubes fitted are positioned outside the danger area.

Danger!

Secure your wheelchair on uneven ground or when transferring (e.g. into a car) by using the brakes.

Danger!

If and whenever possible, during a journey in a specially fitted vehicle for disabled people, vehicle occupants should use the seats in the vehicle and the appropriate restraint system. This is the only way to ensure that occupants will have the maximum protection if there is an accident. When using safety elements offered by SUNRISE MEDICAL and using a specially designed safety system, lightweight wheelchairs can be used as a seat when being transported in a specially fitted vehicle. (See the Chapter on "Transportation").

Danger!

Depending on the diameter and setting of the castors, as well as

the centre of gravity setting of the wheelchair, the castors may begin to wobble at high speeds. This can lead to the castors being blocked and the wheelchair may tip over. Therefore, please make sure that the castors are adjusted correctly (see the Chapter "Castors"). In particular, do not travel on an incline without brakes, travel at a reduced speed. We recommend that novice users use anti-tip tubes.

Danger!

Anti-tip tubes should prevent the chair tipping over backwards unintentionally. Under no circumstances should they take the place of transit wheels, and be used to transport a person in a wheelchair with the rear wheels removed.

Danger!

When reaching for objects (which are in front of, to the side or behind the wheelchair) make sure that you do not lean too far out of the wheelchair, as if you change the centre of gravity there is a risk of tipping or rolling over. The hanging of additional load (back pack or similar items) onto your chair backposts can affect the rearward stability of your chair, especially when used in combination with recliner backrests. This can cause the chair to tip backwards causing injury.

Danger!

For thigh amputees you must use anti-tip tubes.

Danger!

Before setting off, check that your tyre pressure is correct. For rear wheels it should be at least 3.5 bar (350 kPa). The max. pressure is indicated on the tyre. The knee-lever brakes will only work if there is sufficient tyre pressure and if the correct setting has been made (see the Chapter on "Brakes").

Danger!

If the seat and back sling are damaged, you must replace them immediately.

Danger!

Be careful with fire, in particular with burning cigarettes. Seat and back slings can be set alight.

Danger!

If the wheelchair is subject to direct sunlight for a long period of time, then parts of the wheelchair (e.g. frame, legrests, brakes and sideguard) may become hot (>41°C).

Danger!

Always make sure that the quick-release axles on the rear wheels are set properly and lock in. If the button on the quick-release axle is not pressed in, the rear wheel cannot be removed.

Note !

The effect of the knee-lever brake as well as the general driving characteristics are dependant on tyre pressure. The wheelchair is significantly lighter and easier to manoeuvre when the rear wheels are pumped up correctly and both wheels have the same pressure.

Note !

Make sure that your tyres have sufficient tread! Please note that you are subject to any and all traffic laws when driving in public traffic.

Note !

Always be careful with your fingers when working or adjusting the wheelchair!

The products shown and described in this manual may not be exactly the same in every detail as your own model. However, all instructions are still entirely relevant, irrespective of detail differences. The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

Transportation notice for Neon FF (fix front)

Your wheelchair is not suitable for use as a seat in vehicles. Do not sit in the wheelchair during transportation. Use the vehicle seat or a properly secured transport approved seat.

Wheelchair Components (4.1)

We at SUNRISE MEDICAL have been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets relevant National and International Standards to satisfy compliance with EU directives. Optional equipment and accessories are available at an extra charge.

1. Push handle
2. Back sling
3. Sideguard
4. Seat sling
5. Footboard
6. Castor
7. Castor adapter
8. Castor fork
9. Quick-release axle
10. Handrim
11. Rear wheel
12. Cross-brace
13. Brakes

Transportability for Neon SA (swing away)


Transportation of your wheelchair within a vehicle

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security as a vehicle seating system. It is always recommended that the user transfers to the vehicle seating. It is recognised that this is not always practical for the user to be transferred and in these circumstances, where the user must be transported whilst in the wheelchair, the following advice must be followed:

1. Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair, and ensure the method of access/egress is suitable for your wheelchair type. The vehicle should have the floor strength to take the combined weight of the user, the wheel chair and accessories.
2. Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
3. The occupied wheelchair must be located in a forward facing position and secured by the wheelchair tie down and occupant restraint straps (WTORS tie downs meeting the requirements of ISO 10542 or SAE J2249) in accordance with the WTORS manufacturer's instructions.
4. The wheelchair's use in other positions within a vehicle has not been tested e.g. transportation in a side facing position must not be carried out under any circumstances (Fig. a).


There is a risk of serious injury or death if this advice is ignored!

5. The wheelchair should be secured by a Tie Down Restraint system, conforming to ISO 10542 or SAE J2249 with non-adj-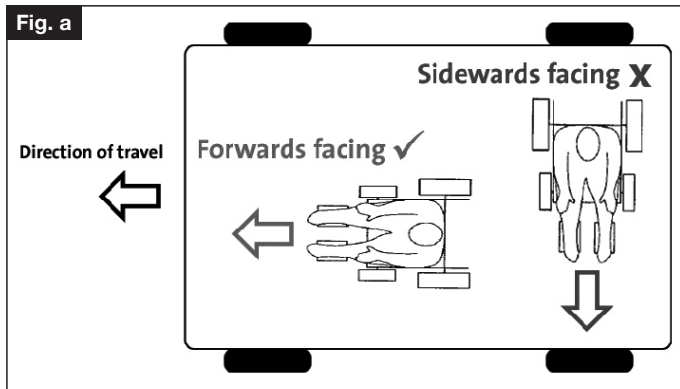

stable front straps and adjustable rear straps, which typically use Karabiner clips/S hooks and tongue and buckle fittings. These restraints generally comprise of 4 individual straps that are attached to each corner of the wheelchair.

6. The tie-down restraints should be fitted to the main frame of the wheelchair as indicated in the diagram on the following page, and not to any attachments or accessories, e.g. not around the spokes of wheels, brakes or footrests.
7. The tie-down restraints should be attached as close as possible at an angle of 45 degrees and tightened securely in accordance with the manufacturer's instructions.
8. Alterations or substitutions must not be made to the wheelchair securement points or to structural and frame or components without consulting the manufacturer. Failure to do so will invalidate the ability of a Sunrise Medical wheelchair to be transported within a vehicle.
9. Both pelvic and upper torso restraint belts must be used to restrain the occupant to reduce the possibility of head and chest impacts with the vehicle components and serious risk of injury to the user and other vehicle occupants. (Fig b) The upper torso restraint belt should be mounted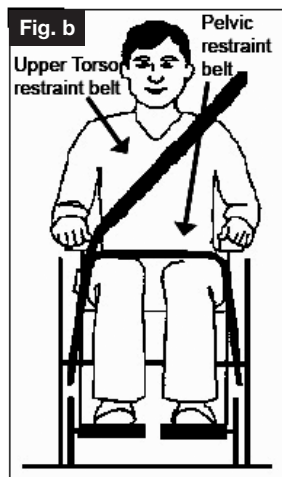


to the vehicle "B" pillar - failure to do so will increase the risk of serious abdominal injuries to the user.

10. A head restraint suitable for transportation (see label on headrest) must be fitted and suitably positioned at all times during transportation.
11. Postural supports (lap straps, lap belts) should not be used or relied on for occupant restraint in a moving vehicle unless they are labelled as meeting the requirements specified in ISO 7176-19:2001 or SAE J2249.
12. The safety of the user during transportation depends upon the diligence of the person securing the tie-down restraints and they should have received appropriate instructions and/or training in their use.
13. Wherever possible remove and stow safely away from the wheelchair all auxiliary equipment, for example:
Crutches
Loose cushions
Tray tables
14. Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported and the wheelchair is restrained using Wheelchair Transport and Occupant Restraints.
15. Reclining backrests should be returned to an upright position.
16. The manual brakes must be firmly applied.
17. Restraints should be mounted to the vehicle "B" pillar and should not be held away from the body by wheelchair components such as armrest or wheels.

Occupant Restraints Instruction



1. The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30 to 75 degrees to the horizontal.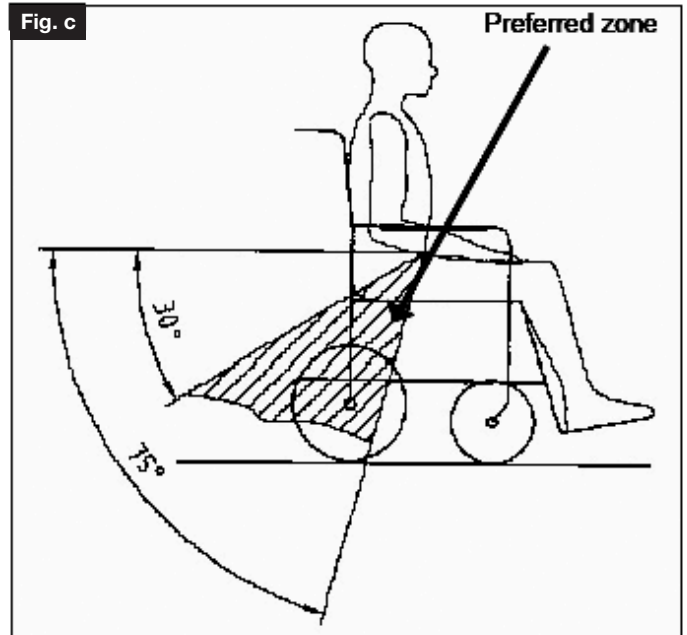


A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75degrees. (Fig c)

2. The upper torso restraint belt must fit over the shoulder and across the chest as illustrated Fig d and e
Restraint belts must be adjusted as tightly as possible consistent with user comfort.
Restraint belt webbing must not be twisted when in use.
The upper torso restraint belt must fit over the shoulder and across the shoulder as illustrated in Fig d and e.

Transportability – positioning of wheelchair tie down restraints on wheelchair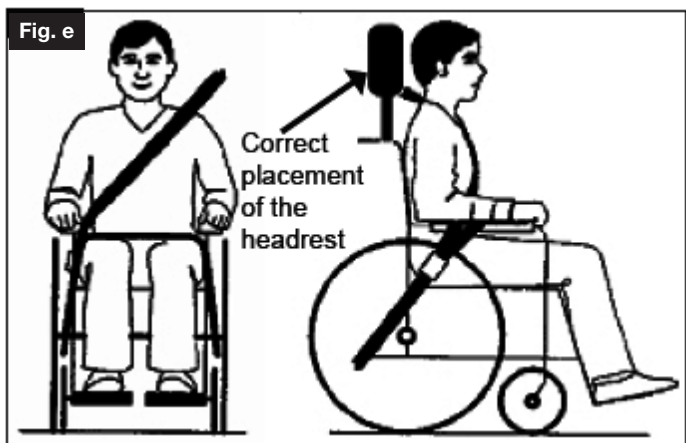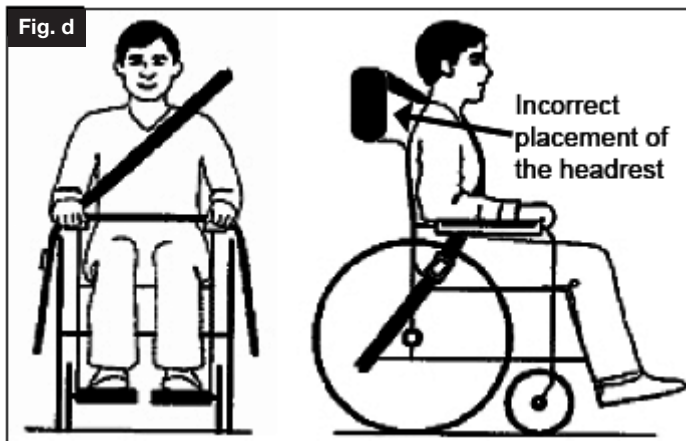

3. The attachment points to the chair are the inner front side frame just above the castor, and the rear side frame. The straps are fitted around the side frames at the intersection of the horizontal and vertical frame tubes.
4. The tie down symbol (Fig f) on the wheelchair frame indicates the position of the wheelchair restraint straps. The straps are then tensioned after the front straps have been fitted to secure the wheelchair.

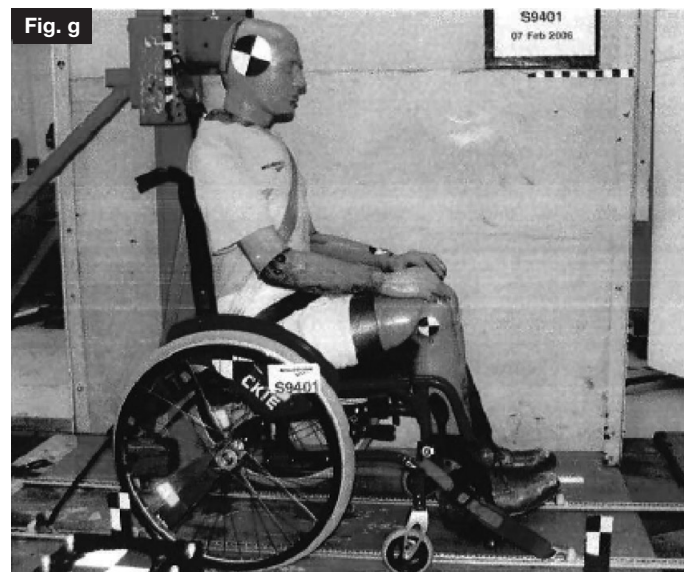The wheelchair secured with front and rear wheelchair tie down restraints (Fig. g).

Positioning of the restraint straps is shown in more detail below:

Position of the front wheelchair tie down restraint and the tie down label (Fig h).

Position of the rear wheelchair tie down restraint and the tie down label on the wheelchair (Fig.i).

Handling

Folding up (6.1)

First remove the seat cushion from the wheelchair and flip up the (platform) footplate or the individual footplates. Take hold of the sling from underneath in the middle and pull upwards. The wheelchair will then fold up. To fold your wheelchair until it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). For this purpose, open the latch from the outside and swing the footrest to the side. Then the footrest only has to be drawn out of the frame tubing.

02.1

02.2

Unfolding

Before unfolding, undo the folding bracket

Press down on the seat tube (see figure). The wheelchair will unfold. Then press the seat tube into the support. This process is easier if you tip the chair a little, so that there is no load on one of the rear wheels and then the chair will unfold more easily.

Caution!

Risk of trapping fingers.

Quick-Release Axles for Rear Wheels (6.2)

The rear wheels are equipped with quick-release axles. The wheels can thus be installed or removed without using tools. To remove a wheel, simply depress the quick-release button on the axle (1) and pull it out.

04

CAUTION:

Hold the quick-release button on the axle depressed when inserting the axle into the frame to mount the rear wheels. Release the button to lock the wheel in place. The quick-release button should snap back to its original position.

Options – Brakes

Wheel Locks (7.10)

Your wheelchair is equipped with two wheel locks. They are applied directly against the tyres. To engage, press both wheel-lock levers forward against the stops (Fig. 282 and 283). To release the wheel, pull the levers back to their original positions.


Braking power will decrease with:

- Worn tyre tread
- Tyre pressure that is too low
- Wet tyres
- Improperly adjusted wheel locks.

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the hand-rims for braking. Make sure that the interval between the tyres and wheel locks complies with given specifications (Fig. 282 and 283). To readjust, loosen screw (1) and set the appropriate interval. Tighten screw (see the page on torque)

520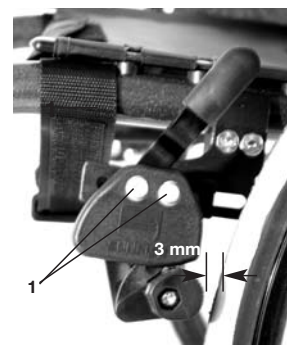

520

CAUTION:

After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.

Extension for the wheel lock Lever (7.11)

The extension for the wheel lock lever can be removed or folded down. The longer lever helps to minimize the effort needed to set the wheel locks.

Caution:

Mounting the wheel lock too close toward the wheel will result in a higher effort to operate. This might cause the wheel lock extension lever to break!

521

Leaning onto the wheel lock extension lever while transferring will cause the lever to break! Splashing water from tyres might cause the wheel lock to malfunction.

Options – Step Tubes

Step Tubes (7.1)

Step tubes are used by attendants to tip a wheelchair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a kerb or step.

Options – Brakes

Drum Brakes (7.13)

Drum brakes permit safe and convenient braking for an attendant. They can also be set with the aid of a locking lever (1) to prevent rolling. This lever must snap audibly into place. Drum brakes are not dependent on the aid of air pressure inside the tyres. Your wheelchair cannot be moved when the drum brakes are set.

12

CAUTION:

Drum brakes should only be adjusted by authorized dealers.

One Hand Wheel Locks (7.16)

Your wheelchair is equipped with two sets of wheel locks, that operate from the left or the right side. They are applied directly against the tyres. To engage, press the wheel-lock lever forward against the stop (fig. 1). To release the wheel, pull the lever back to the original position.

78.2

Holding power will decrease with:

- Worn tyre tread
- Tyre pressure that is too low
- Wet tyres
- Improperly adjusted wheel locks.

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the hand-rims for braking. Make sure that the interval between the tyres and wheel locks complies with given specifications. To readjust, loosen screw and set the appropriate interval.

78.1

Tighten screw (see the page on torque).

CAUTION:


After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.

Mounting the wheel lock too close toward the wheel, will result in a higher effort to operate. This might cause the brake lever extension to break! Leaning on to the brake lever extension while transferring will cause the lever to break! Splashing water from tyres might cause the wheel lock to malfunction.


Options – Suspension System

Suspension system (7.17)

The function of the suspension system is determined by the buffer elements (1). Sunrise Medical offers a varied range of elements depending on the weight of the user.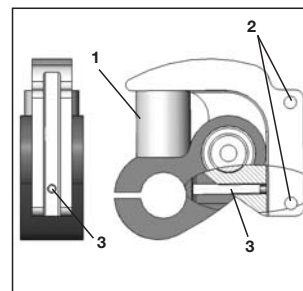

To replace the elements, undo the 2 screws (2). Remove the whole suspension system from the axle plate and undo the internal set screw (3). Turn the rocker arms (3) downwards; the buffer elements (1) can then be easily removed from the upper and lower openings.

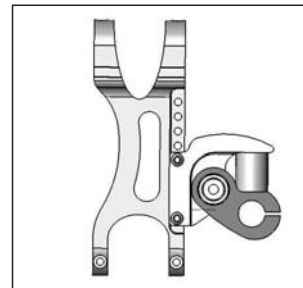To fit new sleeves, use the reverse procedure. Ensure that the buffer elements are securely located in the upper or lower openings.

Options – Footplates

Footboard (7.21)

There are several footboards available. These can be flipped up to make it easier to transfer to/from the chair.

13.1

Lower leg length

By undoing the clamp screw (1), you can adjust the lower leg length. Undo the clamp screw, then adjust the tube with the footboard to the required position.

Tighten clamp screws again (see the page on torque).

Ensure that the plastic push joint is located under the clamp screw in the correct position.

Ensure that you do not go below the minimum gap to the floor of 2.5 cm.

Attention:


When getting into or out of the chair, never stand on the footboards.

Options – Footplates

Divided and Platform Footplates (7.22)

Footplates can be flipped up to facilitate getting in and getting out of your wheelchair.

They can also be tilted to six different angles relative to a level surface. Tighten screw (1) firmly on the outer side. By removing the clips (2), the footplate can be adjusted to three different positions toward both the front and rear. Loosen the adjustment screw (3) to change the horizontal position of the footplate. For this purpose, the footplate must be flipped up. When finished, make sure that all screws have been properly tightened (see the page on torque). A minimum interval of 2.5 centimeters from the ground should always be maintained.

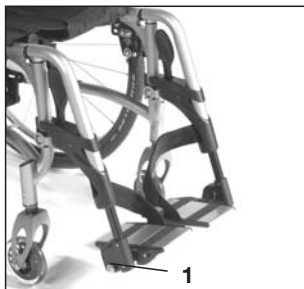15.1

15.2

15.3

Flip-up footrest (standard or with length adjustment): (7.24)

To elevate:

Pull the footrest upwards and adjust it to the desired height. You will hear the footrest click into place.

To lower:

Lift the lower leg and press the release lever to unlock the footrest. You may now lower the footrest. As soon as you let go of the lever, the footrest will lock into place.

17

CAUTION:

Keep hands clear of the adjustment mechanism between the frame and the movable parts of the footrest while elevating or lowering the footrest.

CAUTION:

Footrests are not to be used for lifting or carrying the wheelchair.

Options – Head Rest

Head Rest (7.35)

The headrest can be raised and moved both forwards and backwards. To do this, simply loosen the screw (1 or 2), adjust to the desired position, and tighten the screw (see the page on torque).

309

Options – Footrests and Latch


Footrests and Latch (7.23)

Install your footrest by inserting it with the footplate(s) pointing toward the outside. Then rotate it toward the inside until it locks into place (1). To remove the footrest, depress the latch (2), swing the footrest toward the outside and remove it. Make sure that the footrest is properly locked into place.

CAUTION:

The footrests should not be used to lift or to carry the wheelchair with an occupant.

Little effort is needed to adjust the footrest by means of an eccentric hex nut. Adjustments can be changed as follows: Remove screw. Adjust the eccentric hex nut. Check if the locking mechanism engages and disengages properly. To make fast again, tighten the screw firmly while using a wrench to hold the eccentric hex nut securely (see the page on torque).

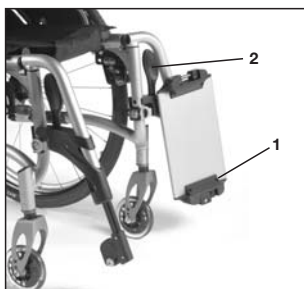16

16

Options – Castor

Adjusting the castor (7.42.3)

To ensure that both forks are set parallel, simply count the teeth visible on both sides.


After setting the castor fork, the teeth will guarantee a secure position, allowing an adjustment of 16° in 2° increments.

Use the flat side to check for a right-angled position to the ground.

The patented design allows the castor fork to be turned, so that it can be reset at right-angles to the ground when the seat angle is changed.

191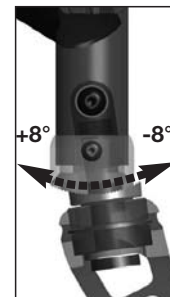

195

Options – Seat height

Adjusting the seat height (7.53.2)

To adjust rear seat height, slacken and remove the 4 Torx screws (A) (2 each side) and the thread washer (B), which secures the clamp (C) for the camber tubes to the axle plates (D). Adjust the two camber tube clamps (C) to the required height and replace the 4 Torx screws. Before tightening the screws, please follow the instructions for setting the toe-in/toe-out to zero (section 7.61.1).

Tighten the screws to 10 Nm.

NOTE – An adjustment to the castor angle may be necessary when adjusting the rear seat height.

Setting the centre of gravity of the seat (7.54.4)

Undo both screws (1) (2 on each side) and make the desired adjustment to the axle adapter on the axle plate. Always make sure that you have selected the same setting on both sides. Tighten the screws to 10 Nm.

CAUTION!

Brakes must be adjusted to suit the new position!

Setting the centre of gravity of the seat with medium and long axle plates (7.54.5)

Undo both screws (3) (2 on each side), and for the NEON with swing-away footrests, also undo the screw (4). Move the perforated plate to the required position. Always make sure that you have selected the same setting on both sides. Insert the screws and tighten to 7 Nm. Then undo the screws under the seat tube and put the seat tube back into the required position. The screws must be tightened to 7 Nm.

CAUTION!

The markings on the right and left seat tube must be in the same position on the rear edge.

Options – wheelbase extension

Wheelbase extension for amputees (7.56.2)

Using the amputee adapter, the wheelbase can be extended by 10.5 cm. Make sure that after any adjustment work, all screws are tightened correctly (see the page on torque).

CAUTION!

Wheel locks must be adjusted to suit the new position!

Options – Wheel tracking

Adjusting the wheel tracking

Fundamentals: To achieve the very best movement, the rear wheels must be adjusted to their optimum setting. To ensure this, the wheel tracking must be adjusted correctly. To do this, measure both wheels to check they are parallel. The difference between both measurements should not be more than 5 mm. By undoing the screws and turning the adapter, you can adjust the wheels to be parallel. Make sure that after any adjustment work, all screws are tightened correctly (see the page on torque).

CAUTION!

Be careful, there is a risk of tipping when adjusting the centre of gravity of the seat!

Options – Back

Angle adjustment of the folding backrest:

1. Open the upper screws and loosen the screw connection (Figure 1).
2. The hole in the back tube must be lined up with the hole in the connecting part, such that you can set the desired backrest angle (Figure 2).
3. Fit the screw connection with the nut and washer until they are hand tight, so that there is no lateral play between the components. But you must still be able to fold the back tube down easily.
4. Set the folding mechanism: To do this, loosen the nuts on the cam slightly (Figure 3). Set the cam (component 1) such that the folding mechanism locks into place with no play (if necessary use a 10 mm open-ended spanner to do this). Then hold the cam in position and tighten the nuts (5 Nm).
5. Repeat steps 1 to 4 for the second side.

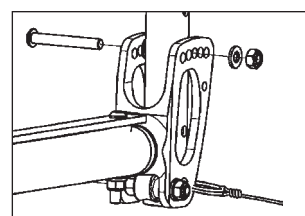
Figure 1

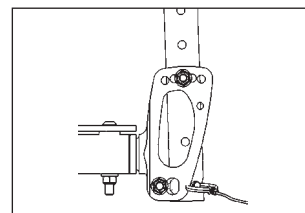
Figure 2

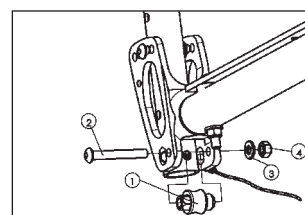
Figure 3

Adjustable back sling (7.77.2)

The adjustable back-sling can be adjusted for tension by using several straps.

The back-sling upholstery can be accessed from the inside via an opening and can be upholstered to suit individual tastes

Adjustable backrest height (7.77.3) 529

The backrest height can be individually adjusted in 2.5 cm steps (3 adjustment ranges: 1 x 25-30 cm, 30-40 cm, 40-45 cm).

Undo the screws (1) and set the backrest to the required height. Then tighten the screws again (see the page on torque).

CAUTION!

Please take into account the altered centre of gravity when setting the angle of the backrest.

Options – Armrests

The front of the armrest is shaped to allow you to pull up close to a desk or table top.

To flip the guard up, push the tab to unlock the armrest.

The height of the armrest can be adjusted by installing the spacers in one of several positions.

To do so, you must first loosen the screws, re configure the spacers and then retighten the screws (see the page on torque).

CAUTION:

Neither the armrest nor the arm-pads are to be used for lifting or carrying the wheelchair.

Standard armrest, Flip-Up, Detachable with desk or full Armrests

Adjust the height of the armrest as follows: Press the tab (1) upward and move to the desired position. Release the tab and push the armpad (2) down until you can hear it lock into place. Always make sure that the armrest has locked into place properly. To flip the armrest up, press the lever (3) to unlock the sideguard.

CAUTION:

Neither the armrest nor the armpads are to be used for lifting or carrying the wheelchair.

CAUTION:

24" rear wheels require the armpad to be re-adjusted by one setting (upwards), to avoid finger entrapment!


Standard sideguard, flip-up, removable with short or long arm pads (7.81.3)

With the sideguard which is stepped down at the front it is possible to move up closer to a table. To flip them up press the lever (1) forwards, so that the sideguard is released.

ATTENTION!

Sideguards, and the armrests are not designed to be used to lift or carry the wheelchair.

Sideguard, flip-up, removable with short or long arm pads, height-adjustable

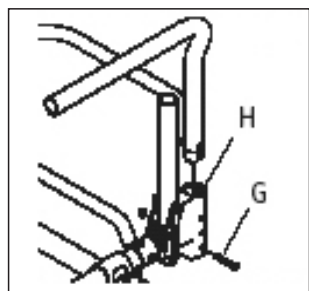530

The arm pad can be height-adjusted in the following way. Pull the lever (1) and adjust the arm pad (2) to the desired height. Release the lever and push the arm pad (2) down until you hear it click into place.

ATTENTION!

Sideguards, and the armrests are not designed to be used to lift or carry the wheelchair.

Options – sideguard

Quickie – height adjustable sideguard (7.81.4)

1. Fitting

- a. Push the outside armrest support into the clamp which is mounted on the chair frame.
- b. The armrest will automatically lock into place.

2. Height adjustment

- a. Turn the release lever to the second position.
- b. Push the arm support up or down to the required height.
- c. Turn the release lever back to the armrest locking position.
- d. Push the arm support in until the upper armrest locks into position.

3. Removing the armrest

- a. Turn the release lever to the first position and remove the armrest.

4. Changing the armrest

- a. Push the armrest back into the clamp.
- b. Turn the release lever back to the armrest locking position.

5. Adjusting the fit of the armrest receiver

- To tighten or slacken the fit of the outside armrest in the clamp:
- a. Release the four bolts on the sides of the clamp.
 - b. Hold the armrest in the clamp and squeeze the clamp together at the required fit.
 - c. Tighten up the four bolts.

6. Adjusting the fit of the inner armrest

- a. The outside armrest is fitted with two stud bolts.
- b. Turn stud bolts in or out as required until you have found the desired fit.

516

Caution!

Sideguards and armrests are not designed to be used to carry the chair.

Sideguard, Aluminum, Bolt-On with Cold-Weather Protection (7.83)

Clothing guards (optional) prevent water from splashing up and soiling clothing. Their position relative to the rear wheels can be set by adjusting the sideguards.

To do this, remove the screws (1 and 2). The armrests (optional) can be adjusted to the desired height after the screws (3) have been removed. When finished, make sure that all screws have been properly tightened (see the page on torque).

CAUTION:

Neither the sideguards nor the armrests are to be used for lifting or carrying the wheelchair.

Options – Lap belt instructions


Before using your wheelchair ensure the lap belt is worn.

The lap belt must be checked on a daily basis to ensure it is free from any obstruction or adverse wear.

Always make sure that the lap belt is correctly secured and adjusted prior to use. Too loose a strap could cause the user to slip down and risk suffocation or cause serious injury.

The lap belt is fitted to the wheelchair as shown in the illustrations. The seat belt comprises 2 halves. They are fitted using the existing seat stay retaining bolt fitted through the eyelet on the belt. The belt is routed under the rear of the side panel. (Fig. 24)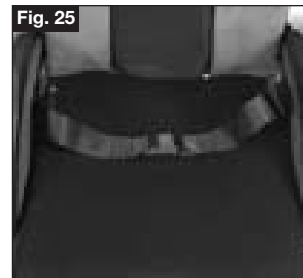

Adjust the belt position so buckles are in the centre of the seat. (Fig. 25)


Adjust lap belt to suit the user's needs as follows:

To reduce the belt length	To increase the belt length
<p>Feed free belt back through male buckle and slide adjusters. Ensure belt is not looped at male buckle.</p>	<p>Feed free belt through slide adjusters and male buckle to provide more belt length.</p>

Options – Push Handles

Height-Adjustable Push Handles (7.96)


These handles are secured with pins to prevent them from sliding out unintentionally. Opening the quick-release lever (1) makes it possible to adjust the push handles to meet your individual needs. As you move the lever, you will hear a locking mechanism; you may now easily position the push handle as desired. The nut (2) on the tension lever determines how tightly the push handles are clamped into place. If the nut is loose after adjusting the tension lever, the push handle will also be too loose. Turn the push handle from side to side before use to make sure that it is clamped securely enough into place. After adjusting handle height, always clamp the tension lever (1) securely into place. If the lever is not secure, injuries could result when ascending stairs.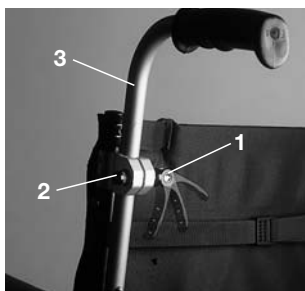

49

Fold-Down Push Handles (7.91)

If the push handles are not in use, they can be folded down by depressing the button (2). When they are needed again, simply flip them back up until they click into place.

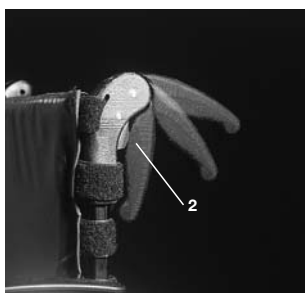
50

When fastened check space between the lap belt and user. When correctly adjusted it should not be possible to insert more than the fat of the hand between the lap belt and the user. (Fig. 26)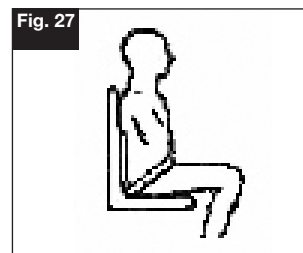

The lap belt should be fixed so that the belt sits at an angle of 45 degrees across the users pelvis. The user should be upright and be as far back as possible in the seat when correctly adjusted. The lap belt should not allow the user to slip down in the seat. (Fig. 27)

	
To fasten buckle: Firmly push male buckle into female buckle.	To release belt: Press exposed sides of male buckle and push towards centre whilst gently pulling apart.

 If in doubt about the use and operation of the lap belt then ask your healthcare professional, wheelchair dealer, carer or attendant for assistance.

Advice to client

The lap belt must only be fitted by an approved Sunrise Medical dealer / agent. The lap belt should only be adjusted by a professional, or a Sunrise Medical approved dealer / agent. The lap belt must be checked on a daily basis to ensure it is adjusted correctly and free from any obstruction or adverse wear. Sunrise Medical does not encourage the transportation of any person in a vehicle using this lap belt as a method of restraint. Please see Sunrise Medical transit booklet for further advice on transportation.

Maintenance:

Check lap belt, and securing components, at regular intervals for any sign of frays or damage. Replace if necessary.

NOTE:

The lap belt should be adjusted to suit the end user as detailed above. Sunrise Medical recommend that the length and fit of the belt be checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length.

Options – Anti-Tip Tubes

Anti-Tip Tubes (7.93)

Anti-tip tubes (1) provide additional safety for inexperienced users when they are still learning how to operate their wheelchairs. Anti-tip tubes (1) prevent a wheelchair from tipping over backwards. Pushing the anti-tip tubes (1) will move them downwards; they can also be swung forward. An interval of 3 cm to 5 cm should be maintained between them and the ground. You must swing the anti-tip tubes forward when going up and down large obstacles (such as a kerb) to prevent them from touching the ground.


53

Options – Standard Sling

Standard Sling (7.101)

The sling is equipped with a Velcro fastener on one side; this permits stepless adjustment of the sling. To readjust the sling, first fold up the wheelchair slightly. Then remove the screws (1) and pull the front plugs (2) forward out of the frame. The sling (3) can then be slid off of the frame. By unfastening the Velcro, the sling can be readjusted. Reverse this procedure to reinstall the sling. When finished, make sure that all screws have been properly tightened (see the page on torque).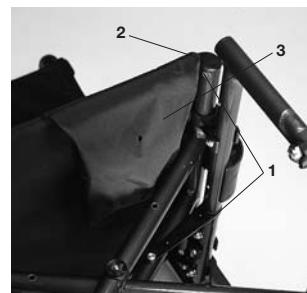

54

CAUTION:

For proper safety, at least 50 % of the Velcro surfaces must make contact with each other at all times.

Options – Crutch Holder

Crutch Holder (7.110)

This device permits crutches to be transported directly on a wheelchair. It has a Velcro loop (1) to fasten crutches or other aids.

CAUTION:

Never try to use or even remove the crutches or other aids while moving.

306

Options – Tray

Tray (7.111)

The tray provides a flat surface for most activities. Before using a tray, it first has to be adjusted to the width of the seat by an authorized dealer. The user must be sitting in the wheelchair whilst this adjustment is made.

60

Options – Stabilising bar

Stabilising bar (7.112)

The stabilising bar gives the backrest its stability. To be able to fold the chair, you must turn the knobs (1) on the end of the push handle and remove the stabilising bar, then you can fold the wheelchair easily. To replace the stabilising bar, you must retighten the knobs, so that it is locked in place.


61

Transit Wheels

Transit wheels (7.113.1)

1. Removing the transit wheels from the clamps (Fig. 532)
 - a. Pull out the release screw (D).
 - b. Remove the transit wheel (E) from the clamp (F).
 - c. Repeat this procedure on the other side.
2. Fitting the transit wheels into the clamps (Fig. 532)
 - b. Slot the transit wheel (E) into the clamp (F).
 - b. Align the holes into position for the appropriate rear wheel size.
 - c. Replace the release screw (D).
 - d. Repeat this procedure on the other side. Both transit wheels must be set at the same height.
3. Adjusting the transit wheels to the frame angle (Fig. 532). The alignment of the transit mounting must be adjusted as required to suit the angle of the frame rails. The castor mountings come with two hole patterns which offer 4 positions (Fig. 532). Adjusting the fitting configuration:
 - a. Remove the fitting screw (D).
 - b. Turn the transit mounting on to the hole pattern which gives the smallest position. The castor must always be located behind the mid line of the axle.
 - c. Insert the fitting screw (D) and tighten to 12 Nm.
 - d. Repeat this procedure on the other side. Both transit wheels must be set at the same angle.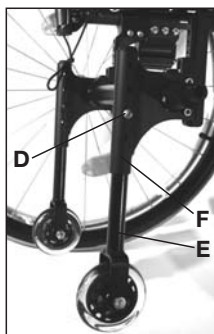

532

Wheelchair wobbles (9.7)

- Check angles of castors
- Check tyre pressure
- Check whether the rear wheels have been adjusted differently

Maintenance and Care

Maintenance (9.9)

- Check the tyre pressure every 4 weeks. Check all of the tyres for wear or damage.
- Check the brakes approximately every 4 weeks to make sure that they are working properly and easy to use.
- Change tyres as you would an ordinary bicycle tyre.
- All of the joints that are critical to using your wheelchair safely are self-locking nuts. Check every three months to make sure that all bolts are secure (See the page on torque). Self locking nuts should only be used once and should be replaced after single use.
- Use only mild household cleansers when your wheelchair is dirty. Use only soap and water when cleaning the seat upholstery.
- If your wheelchair should ever get wet, please dry it after use.
- A small amount of sewing-machine oil should be applied to quick-release axles approximately every 8 weeks. Depending on the frequency and type of use, we recommend taking your wheelchair to your authorised dealer every 6 months to have it inspected by trained personnel.

CAUTION:

Sand and sea water (or salt in the winter) can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure to these conditions.

Hygiene when being reused:

When the chair is to be reused, it should be prepared carefully, and be wiped and treated with spray disinfectant on all surfaces which could come into contact with the user.

In doing this, you must use a disinfectant taken from the DGHM list, e.g. Antifect Liquid (Schülke & Mayr) for alcoholic quick disinfection for medical products and medical inventory when you require a quick effect.

Please pay attention to the manufacturer's instructions of the disinfectant you are using.

In general, a safe disinfectant cannot be guaranteed on seams. We therefore recommend, that you properly dispose of seat and back slings in the case of microbacterial contamination with active agents according to §6 infection protection law.

Tyres and Mounting

Tyres and Mounting (8.1)

Always make sure that you maintain the correct tyre pressure, as this can have an effect on wheelchair performance. If the tyre pressure is too low, rolling resistance will increase, requiring more effort to move the chair forward. Low tyre pressure also has a negative impact on manoeuvrability. If the tyre pressure is too high, the tyre could burst.

The correct pressure for a given tyre is printed on the surface of the tyre itself.

Tyres can be mounted the same way as an ordinary bicycle tyre. Before installing a new inner tube, you should always make sure that the base of the rim and the interior of the tyre are free of foreign objects. Check the pressure after mounting or repairing a tyre. It is critical to your safety and to the wheelchair's performance that regulation air pressure be maintained and that tyres be in good condition.

Trouble Shooting

Wheelchair pulls to one side (9.1.1)

- Check tyre pressure
- Check to make sure that the wheel turns easily (bearings, axle)
- Check to make sure both castors are making proper contact with the ground.

Wheelchair does not fit in the support (9.3)

- The chair is still new, or the seat or back slings are still stiff, this will ease through frequent use

Wheelchair is difficult to fold up (9.5)

- Adjustable backrest sling is too stiff, loosen slightly.

Wheelchair squeaks and rattles (9.6)

- Check to make sure all bolts are secure; tighten if necessary (see the page on torque)
- Apply small amount of oil to moving parts

Nameplates/Guarantee

Nameplates (11.1.2)

The nameplate is located on either the cross-tube assembly or the transverse frame tube, as well as on the back page of the user's manual. The nameplate indicates the exact model designation and other technical specifications. Please provide the following information whenever you have to order replacement parts or to file a claim:

- Serial number
- Order number
- Month/Year

Guarantee

You have purchased a high-quality SUNRISE MEDICAL product. As a sign of our gratitude, we are providing you with 5-year guarantee on all frame components and the cross-tube assembly. We are not responsible for any damage resulting from inappropriate or unprofessional installation and/or repairs, through neglect and wear, or from changes in any wheelchair components caused either by the user or by third parties. In such cases, this guarantee shall be considered null and void.

Custom wheelchairs cannot be exchanged.

Technical Data

Seat widths: 30-50 cm in 2 cm steps
 Seat depths: 30-50 cm in 2 cm steps

Seat heights:

Castors	Fork	Front seat height in cm	Rear seat height in cm		
			22"	24"	26"
3" solid	72 mm	43	33-43	43-35	43-42
	72 mm	44	33-44	44-35	44-42
	72 mm	44	33-44	44-35	44-42
4" solid	118 mm	46	34-45	46-35	46-42
	118 mm	47	35-45	47-35	47-42
	118 mm	48	36-45	48-36	48-42
	118 mm	49	38-45	49-37	49-42
	118 mm	50	38-45	50-39	49-42
	138 mm	48	36-45	48-36	48-42
	138 mm	49	38-45	48-37	49-42
	138 mm	50	38-45	48-39	49-42
	138 mm	51	39-45	48-39	49-42
	138 mm	52	40-45	48-40	49-42
5" solid	118 mm	49	33-45	48-37	49-42
	118 mm	50	38-45	48-39	49-42
	118 mm	51	39-45	48-39	49-42
	138 mm	50	38-45	48-39	49-42
	138 mm	51	39-45	48-39	49-42
	138 mm	52	40-45	48-40	49-42
6" soft	118 mm	51	39-45	48-39	49-42
	118 mm	52	40-45	48-40	49-42
	138 mm	51	39-45	48-39	49-42
	138 mm	52	40-45	48-40	49-42
	138 mm	53	41-45	48-41	49-42
	138 mm	54	42-45	48-42	49-42










Overall length:
 107 cm with seat depth of 50 cm

Overall height:
 97 cm with back height of 45 cm

Weight in kg:
 from 9.9 kg

Maximum load:
 If the seat width is 42 cm, then 115 kg, above this it is 140 kg

Torque (12.1)

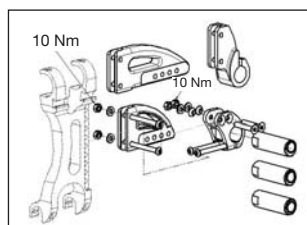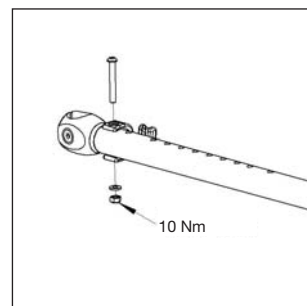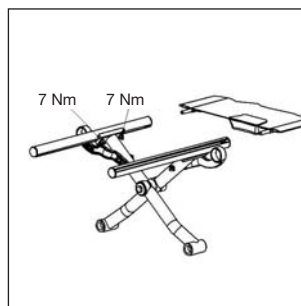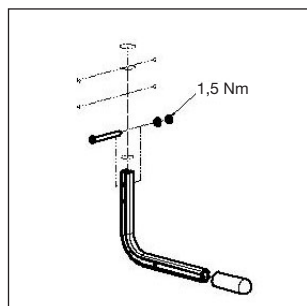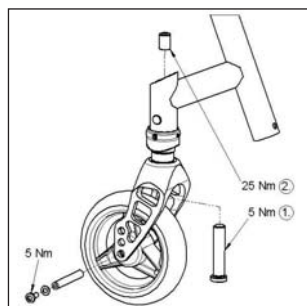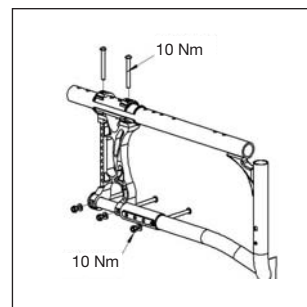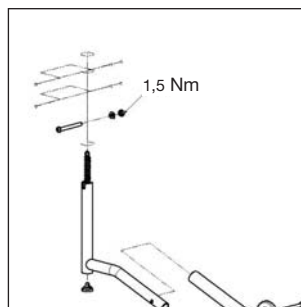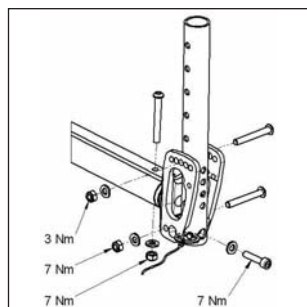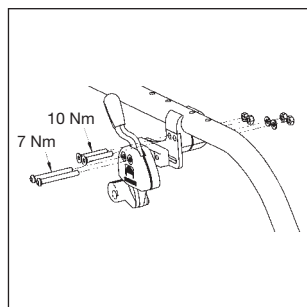Torque (12.1)
 The torque for the M6 screw is 7 Nm, unless otherwise specified.

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Sunrise Medical AB
Sunrise Medical B.V.
G2 A/S