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PARAGUAS BABY CARRIDGE FOR CHILDREN

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Paraguas baby carridge A01 / A02 / A03 / A04 / A10

The Nido Baby carriage has been designed to be used by physically or mentally handicapped children for baby carriage.

This chair is not valid for deformity treatments.

Net weight admissible: 50 kg. (75 kg. A10/1, A10/2).

Using the baby carriage as a seat into vehicles is forbidden.

The prolonged use of this baby carriage can be produce irritation skin to the user. To avoid this, exists a comfortable cover as an accesory.

Be careful with the environment. This baby carriage must be disposed of in compliance with local waste disposal laws.

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General characteristics

Adjustable back in two positions for A01, A02, A03 and A04.

Fixed back in one position in A10.

Fire resistant upholstery in compliance with laws.

Lateral Belts for securing.

Foot rest adjustable in several positions.

Several progressive sizes according to user size.

Guarantee: According to the current legislation.

Guarantee will be rescind if an inappropriate use, handling of components and careness to this use handbook.

Excluded are: damages from corrosion due to abnormal environmental condition such as humidity and salinity.

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This product is in compliance with the requirements of the European Directive 93/42/CEE of sanitary products

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UNFOLD/FOLD

When the baby carriage is folded, get free the lateral hook (1) and, leaning on the lateral structure and the handlebars, unfold the baby carriage (2). After, block the pieces (3) and (4) to get the baby carriage ready for use. To fold the baby carriage, follow the previous steps in reverse order. A10, A10/1, A10/2 don't forget to fix the lateral block system.

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BACK POSITIONS (A01, A02, A03, A04).

The back can be graduated in two points, moving the lateral mobile pieces (5) till their maximums. **IMPORTANT:** Be sure the two mobile pieces are correctly positioned and coincide to the colour marks.

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FOOT REST POSITIONS

Over the pipe of the forward wheels are the lateral blocks (6) of the foot rests. They can be graduated in several heights. **ATTENTION:** Do not to fold the baby carriage when the foot rest are backed.

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USING THE BABY CARRIDGE

To transfer the user, brake the baby carriage in a not slipping floor. Incorporate the user till the seat be empty. To go down or to climb a border or step, lean the whole weight over the back wheels and advance carefully. Be sure the back wheels are correctly engaged on its use position. (A22 and A23). Its recommended using the adjustable seat belts.

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CLEANING

Structure and upholstery: Clean whit water and neuter soap. To clean the baby carriage, the upholstery can be extracted, removing all the buckles and belts.

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MAINTENANCE AND REPAIRING

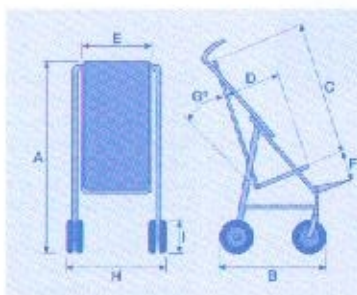
This baby carriage doesn't need maintenance. In case of fault or repairing, contact with an approved retailer

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ACCESORIES

Waterproof hood according to size A34. Comfortable cover for seat and back according to size A36.

	A01	A02	A03	A04	A10	A10/1	A10/2
A	810	1.020	1.020	1.030	955	990	990
B	650	755	740	740	665	690	690
C	505	560	620	820	530	530	570
D	270	300	350	300	305	380	400
E	320	360	400	420	395	390	440
F: min./max.	250/360	220/280	290/320	310/370	270/400	290/440	290/440
G	95°/120°	90°/115°	90°/100°	90°/100°	-	-	-
H	470	530	570	570	530	610	640
I	175	175	175	175	175	300/200	300/200
Kg.	8,4	8,9	10,6	11,5	8,7	15	15

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Measures in milimetres